

HYPAFIX® TRANSPARENT

NON-STERILE FILM ROLL FOR WATERPROOF FILM DRESSING FIXATION


The skin-friendly, highly breathable wide area fixation film is designed to protect reliably against water, bacteria and contaminants. The innovative application technology offers an intuitive design with a distinctive red stripe for fast, easy and accurate placement.



Easy application with distinctive red strip




1 Cut **Hypafix®** transparent to the required size using the grid pattern as a guide.

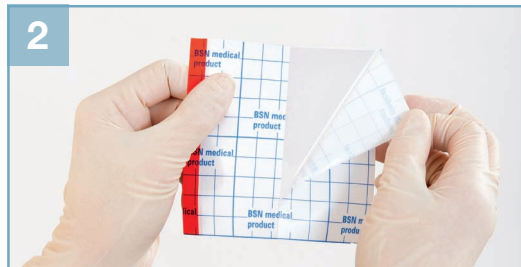

2 The release paper is in two parts. First remove the part without the red strip.



3 Place the film over the wound dressing and fix it onto the skin carefully. Fold back the remaining part ...

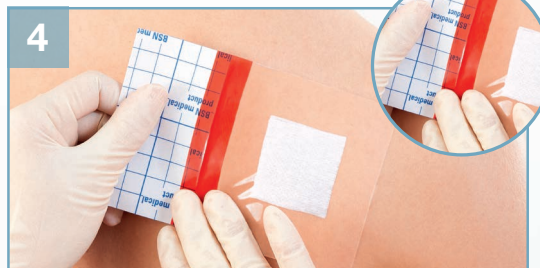

4 ... and pull it across the wound dressing. Smooth the film in place and press down well over the red strip.



5 Remove the remaining release paper with the application film in a single movement.



6 To painlessly and gently remove it, stretch the dressing at one corner and pull it towards you.